


Westerstrand Urfabrik AB

Time Log Manual

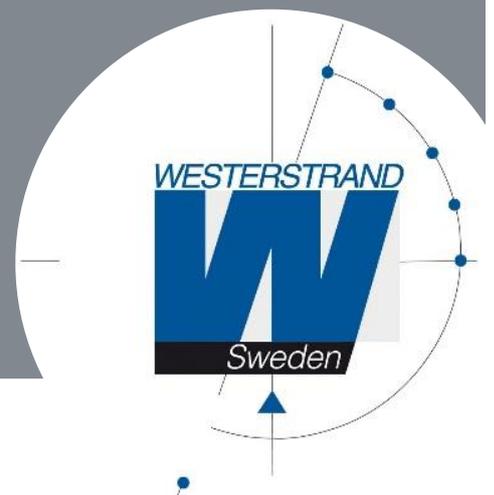

Fourth generation




We have all time for you

Description

Westerstrand's Time Log visualizes and makes it easier to understand the abstract concept of time. When the time passes, the diodes in the timeline go out one by one until the time is up and the timer sounds. The Time Log illustrates the concepts of time such as WHEN or HOW LONG daily routines and activities should take place. Regular use of the log can help you develop the perception of time and reduce stress and frustration in everyday life.

The Time Log is dust and waterproof and can be used indoors, outdoors and in the shower. If the Time Log is used when bathing, our protective bag containing a floating cushion is also available as an option. Using the protective case allows the hour log to float.

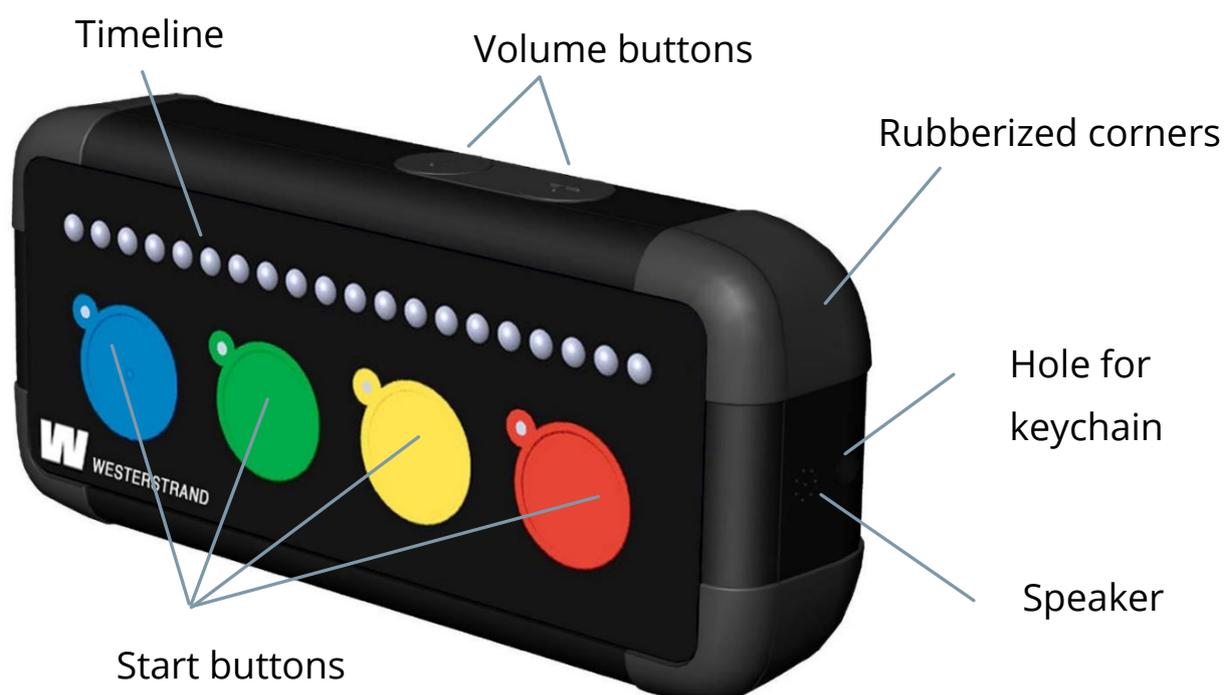

Start up

Set the time

Press one of the start buttons (red, yellow, green or blue) corresponding to the desired time. The LEDs, light up and the countdown starts. Next to each start button, a small diode also lights up to show the selected time.

Volume

Press the buttons on the top to increase or decrease the volume of the alarm signal. The volume has 8 different modes. When highest or minimum volume is reached, the Time Log vibrates for 1 second.

Press **+** to raise the volume

Press **-** to lower the volume

For silent mode hold **-** pressed for 1,5 sec.

For highest volume hold **+** pressed for 1,5 sec.

Function

The countdown is visualized by the red LEDs going out, one by one, as time passes. To indicate that the selected time has expired, the Time Log alarms with flashing diodes, an audible signal and vibration for a few seconds. The started time can be deleted by simultaneously pressing the red and blue start button for 2 seconds and then releasing it. After that, a new time can be started.

Hidden functions

Adjust brightness and Activate/deactivate vibration

1. Press on one of the start buttons (red, yellow, green, or blue) to activate the Time Log.
2. Press the green and yellow buttons at the same time for about 2 seconds until you hear a "up sound".
3. The 20 red LEDs light up at the current brightness. Green and yellow LEDs also light up.

Press **+** to increase the brightness.

Press **-** to lower the brightness.

4. Continue by pressing the green or yellow button.
5. A beep is heard, and the red LEDs starts to "wander".

Press **+** to activate vibration – a beep is heard, and the Time Log vibrates.

Press **-** to deactivate vibration – a beep is heard and after a few seconds another beep is heard.

6. Continue by pressing the green or yellow button. A "down sound" is heard and the LEDs go out. The settings are now saved.

(If no button presses/selections are made within about 1 min, the old settings are restored, and the timer goes into sleep mode)

Hidden functions

Lock/unlock buttons

1. Press on one of the start buttons (red, yellow, green, or blue) to activate the Time Log.
2. Press on all start buttons (red, yellow, green, and blue) at the same time for about 2 seconds until a “down sound” is heard.
3. Release the buttons to activate or deactivate the lock function.
(The lock function is automatically deactivated after the time has expired)

Restore factory settings

1. Press on one of the start buttons (red, yellow, green, or blue) to activate the Time Log.
2. Press on all start buttons (red, yellow, green and blue) at the same time for about 3 seconds until **two** “down sound” is heard.
3. Release the buttons. The Time Log will play some “up- and down sounds”.
4. When the LEDs have gone out, a factory reset has been done.
(Brightness approx. 70% of maximum and vibration is activated)

Battery operation

The Time Log is powered by two batteries. Batteries are included at delivery. These are enough for about 60 hours of use before they need to be replaced. When the light of the LEDs starts to fade, both batteries must be replaced with new ones. Remove the batteries if you are not going to use the Time Log for an extended period.



Locking screw for battery cover

Locking screw for battery cover

Change batteries like this:

1. Unscrew the battery cover locking screws with a small Phillips screwdriver
2. Replace the batteries with new AA batteries. Make sure you put them in the right direction, + and -
3. Screw the battery cover back on. Note that the battery cover only fits one way.

Technical specification

The box contain

1 x Time Log (Incl. 2 x AA batteries)

1 x Keychain

1 x Manual

Size	105x47x28 mm
Weight	75,2 g (excl. batteries)
	121 g (incl. batteries)
Power	2 pcs 1,5V R6 / AA-batteries
Ambient temperature	-10°C to +40°C
Protection class	IP67 (dust and waterproof)

Start button-colours	Time Log models				
	8	20	60	80	8
	2 min	5 min	15 min	20 min	2 h
	4 min	10 min	30 min	40 min	4 h
	6 min	15 min	45 min	60 min	6 h
	8 min	20 min	60 min	80 min	8 h

Maintenance

Cleaning

When cleaning, wipe the Time Log with a damp and lint-free cloth with a normal cleaning agent, alternatively with a cleaning cloth intended for electronics.

Environment & Warranty

We provide a 1-year warranty for manufacturing defects. The warranty does not cover handling errors.



The Time Log contains electronics and must therefore be recycled as electrical waste. This is normally left at your recycling centre, contact the municipality if you are unsure.

Security

Hole for Keychain



On one side of the Time Log there is a hole where you can attach the keychain accessory to wear the Time Log around your neck. Use only the supplied original accessories for your time log to avoid the risk of suffocation and other injuries.

The product must not be used in any other environment or in any other way than prescribed.

Batteries contain hazardous substances. Store batteries out of reach of unauthorized persons.

Contact Westerstrand immediately in the event of damage or if there is a risk of damage.

The latest version of the manual can be downloaded from westerstrand.se

Manufacturer of Swedish quality products since 1906



Westerstrand Urfabrik AB
Verkstadsgatan 1
SE- 545 31 Töreboda



info@westerstrand.se



+46 506 480 00



westerstrand.se



Westerstrand Urfabrik AB
Tid- & Resultatanläggningar



Westerstrand Urfabrik AB

DOK11084en02