

Time Log

Visual time aid



***New version** of our well-known Time Log. Our visual time aid has undergone a series of improvements and updates.*

***The fourth** generation of the Time Log is now ready to enter the market.*






Westerstrand's Time Log visualizes and makes it easier to understand the abstract concept of time. When the time passes, the diodes in the timeline go out one by one until the time is up and the timer sounds. The time log illustrates the concepts of time such as WHEN or HOW LONG daily routines and activities should take place. Regular use of the log can help you develop the perception of time and reduce stress and frustration in everyday life.

Description

The product is developed in collaboration with scientists specializing in technical aids. There are now nine different versions that cover time periods from 2 minutes up to 8 hours. Each log version has 4 pre-programmed time periods to choose from. See all variants on the last page. All variants have rubberized corners for extra shock absorption and protection against slipping.

The delivery includes a keychain and instructions for use. The log is powered by two AA batteries. Batteries included in delivery. These are enough for about 60 hours of use before they need to be replaced. The log is dust and waterproof and can be used indoors, outdoors and in the shower. If the log is used for swimming, there is also as an option, our protective bag that contains a floating pillow. Use of the protective case allows the log to float.

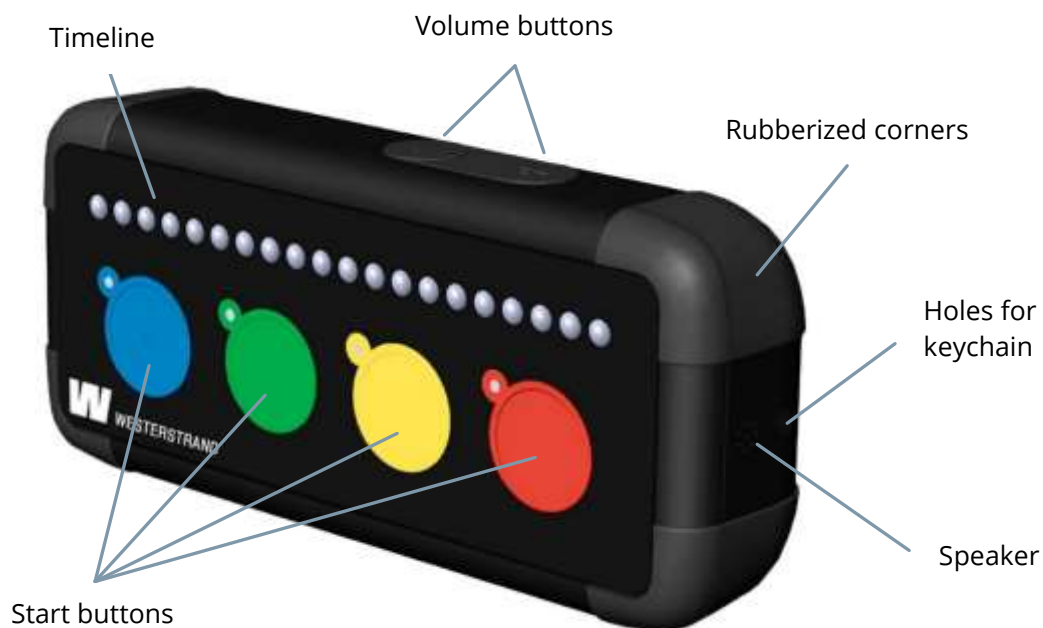



Technical specification

Size	105x47x28 mm
Weight	75,2 g (excl. batteries)
	121 g (incl. batteries)
Power	2 pcs. 1,5V R6 / AA-batteries
Ambient temperature	-10°C till +40°C
Protection class	IP67 (dust and waterproof)

Function

By pressing one of the start buttons (red, yellow, green or blue) with the selection of the current time period number that corresponds to the desired time, the number of LEDs that the start button represents lights up and the countdown starts. Next to each start button is a small diode that lights up to display the selected time. The started time can be deleted by pressing and holding the red and blue start button simultaneously for two seconds and then releasing. Then a new time can be started.

The countdown is visualized by a series of red LEDs that go out, one by one, as time goes on. To indicate that the selected time has elapsed, the timer alarms with flashing diodes, beep and vibration. The display lasts for a few seconds but can be switched off by pressing one of the start buttons. The sound level is easily regulated via the + and - buttons on the upper long side of the log.





Article numbers

Article number	Version	
119628-02	8 min	
119622-02	20 min	 
119622-12	20 min - no numerals	
119622-22	20 min standing	
119620-02	60 min	 
119620-12	60 min - no numerals	
119623-02	80 min	 
119623-12	80 min - no numerals	
119630-02	8 hours	

